



# **INSTRUCTION MANUAL**

THANK YOU FOR PURCHASING CAR CANE. WE HOPE YOU ENJOY IT!

Please read instructions carefully.

- Non-slip grip is comfortable to hold.
- Gives you the leverage to lift yourself up.
- Stores away easily in your door or glove box.
- Built-in flashlight, seat belt
  - cutter and window breaker.



Insert Car Cane<sup>™</sup> into car door latch.



The Car Cane<sup>™</sup> will hook Hold onto Car Cane<sup>™</sup> to firmly in place.



support yourself when getting out of your car.

#### CAR CANE<sup>™</sup> BONUS FEATURES



LED Flashlight



Seat Belt Cutter



Window Breaker





## **INSTRUCTION MANUAL**

THANK YOU FOR PURCHASING CAR CANE. WE HOPE YOU ENJOY IT!

Please read instructions carefully.

### REPLACING BATTERIES FOR LED FLASHLIGHT

1. Remove small screw on the side of Car Cane<sup>™</sup> using a small Phillips screwdriver. See Diagram on right.

2. Remove the battery compartment cover by sliding it out as shown on right.


3. Insert 2 x 3V Lithium CR2032 batteries. Observe polarity.


4. Replace the cover, insert screw and tighten with screwdriver.

#### TO SEE OUR FULL RANGE OF PRODUCTS VISIT:

www.highstreettv.com



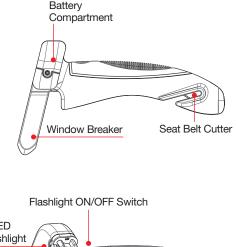

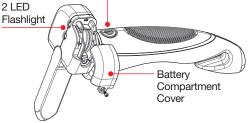

Distributed by: High Street TV, PO Box 724, Altrincham, WA15 5BJ